



Shade	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
<b>Liner</b>	1	2	2	2	4	1	1	2	2	1	3	3	4	1	5	5
<b>Titan Opaquer</b>	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
<b>Core Dentin</b>	CD2	CD6	CD6	CD3	CD5	CD6	CD6	CD3 CD5 <sup>1)</sup>	CD5	CD2	CD4		CD5	CD4		CD5
<b>Opaque Dentin</b>	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
<b>Dentin</b>	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
<b>Enamel</b>	LIGHT YELLOW	YELLOW	YELLOW	DARK ORANGE	DARK ORANGE	LIGHT YELLOW	YELLOW	ORANGE	DARK ORANGE	YELLOW	YELLOW	ORANGE	DARK ORANGE	LIGHT YELLOW	YELLOW	ORANGE
<b>Enamel<sup>2)</sup></b>	57	57	58	59	59	57	58	58	59	58	59	59	59	59	59	59



1) CD3 | CD5 in a 50:50 ratio  
2) The Enamel masses 57-59 are used in combination with Core Dentin.